

# Great Lakes NOLA Protraction Headgear



## Face Mask Headgear

300-306 Small

300-307 Large



## Hickham Headgear

300-308 Small

300-309 Large

ORDERING INFORMATION ON BACK ►

## NOLA Protraction Headgear Advantages:

- NOLA Face Mask Headgear is easy to fit and has a comfortable headstrap.
- Hickham Headgear is comfortable for use while sleeping, does not inhibit mandibular movement, and is also ideal for daytime wear.
- NOLA Protraction Headgear are designed to bring individual teeth, groups of teeth, or entire arches forward. The headgear is fully adjustable and offers two chin cup sizes.

## Treatment Suggestions:

Elastics attached to the terminal teeth will act like Class III elastics to the upper arch. In other words, they will tend to open the bite by extruding these teeth. It is therefore desirable to have the elastics attached to the anterior part of the arch. This is especially true in Class III cases where the premaxilla needs to be developed down and forward in a normal growth pattern.

Occasionally, the patient will perspire excessively for the first three days the chin cup is worn. The tissue will adapt and the perspiring usually stops. Do not try to remedy the problem by drilling holes in the cup for air circulation. Air will not circulate through the holes and the patient will not like the spotted chin appearance when the cup is removed. If perspiring persists, or the patient is allergic to acrylic, cloth liners are available.

**IMPORTANT REMINDER FOR DOCTORS!** To avoid risk of eye injury, be sure to inform patients and parents about proper safety precautions when placing and removing headgear. Also caution patients against rough or risky play while headgear is in place.

ITEM	ORDER NO.
<b>Protraction Headgear &amp; Replacement Parts</b>	
Small Face Mask Large Face Mask	300-306 300-307
Small Hickham Large Hickham	300-308 300-309
Headstrap for Hickham (both sizes)	300-312
Forehead Strap and Lock Nuts for Face Mask (both sizes)	300-313
Lock Nuts for Forehead Strap (2/pkg)	300-314
Plastic Protector Tips (10/pkg)	300-316



< SCANNEZ LE QR CODE POUR ACCÉDER AUX INSTRUCTIONS EN FRANÇAIS  
 < ESCANEE EL CÓDIGO QR PARA ACCEDER A LAS INSTRUCCIONES EN ESPAÑOL  
 < SCANSIONA IL QR CODE PER ACCEDERE ALLE ISTRUZIONI IN ITALIANO  
 < SCANNEN SIE DEN QR-CODE, UM ZU DEN DEUTSCHEN ANLEITUNGEN ZU GELANGEN  
 < SCAN QR-KODEN FOR AT FÅ ADGANG TIL VEJLEDNINGEN PÅ DANSK  
 < SCAN DE QR-CODE OM TOEGANG TE KRIJGEN TOT DE INSTRUCTIES IN HET NEDERLANDS



**Great Lakes®**  
 DENTAL TECHNOLOGIES

200 Cooper Avenue, Tonawanda, New York 14150 USA  
 greatlakesdentaltech.com | 800.828.7626  
 Great People. Great Products. **Great Lakes.**