

Nylon Appliance Cleaning Instructions

In order to keep your nylon appliance in optimal condition, we recommend you clean it as follows:

> Daily - After Each Wear

- Immerse the appliance in lukewarm water.
- Clean the nylon appliance with a denture brush. Use the smaller end of the brush to clean the inner surface for at least one minute. Then, switch to the larger brush and clean the outer surface for an additional minute.
- Rinse well with clean, lukewarm water.
- Inspect the appliance to confirm it is visibly clean. If necessary, repeat the cleaning process.
- Once clean, dry the appliance with a clean tissue or towel, and place it back in its container.

> Twice a Week - Deep Clean

To clean your nylon appliance more thoroughly, use DentaSOAK®.

- Read the DentaSOAK instructions before use.
- Clean the nylon appliance following the DentaSOAK instructions.
- Rinse the appliance with lukewarm water.
- Once clean, dry the appliance with a clean tissue or towel, and place it back in its container.

[More ►](#)

Nylon Appliance Cleaning Instructions

> Caution

- Do Not put your appliance in water warmer than 45°C / 113°F.
- Do Not use toothpaste on your appliance as it may be too abrasive.
- Do Not use chlorine-based products, bleach, acetic acid, household cleaning products, or alcoholic solutions on your nylon appliance.
- Do Not use mouthwash to clean your device, this could stain and induce odor to taste.

For DentaSOAK cleaning supplies, visit:
Dentasoak.com

Consult your doctor with any questions or concerns.



Great Lakes[®]
DENTAL TECHNOLOGIES

GREAT PEOPLE. GREAT PRODUCTS. GREAT LAKES.