


## ROD DIMENSION GUIDE / GUIDE DE MESURE

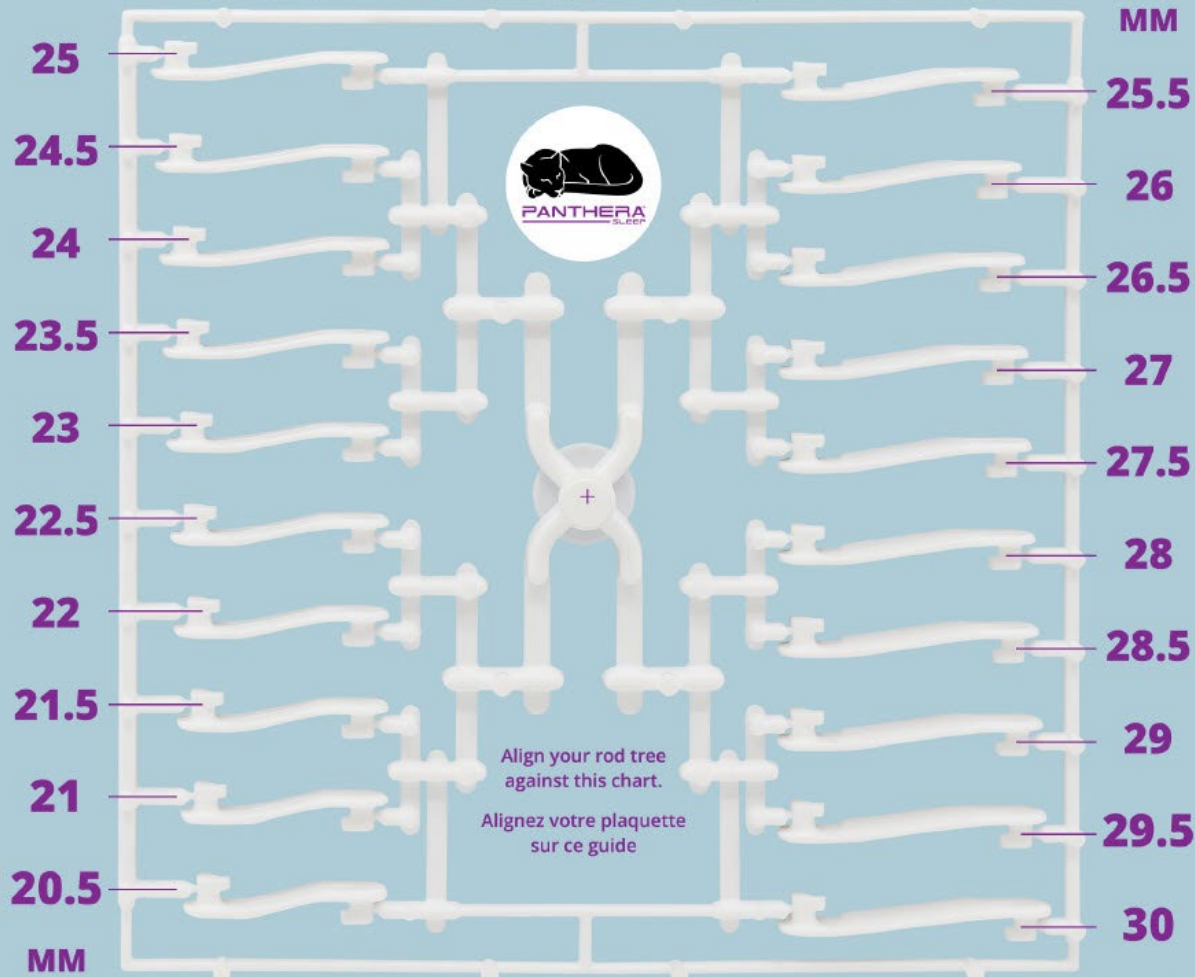

### INSTRUCTIONS FOR USE:

The rods removed from the trees are the starting rods on the Classic (D-SAD).

The number carved in the rod indicates its length in mm. **Always use the same length of rods on both sides.**

**Shorter rods increase protrusion**  
**Longer rods decrease protrusion**

You will not need all the rods of the tree pack.

The dentist may retain the tree rod pack and supply rods gradually.

It is recommended:

- to not adjust in increments of more than 1mm at a time
- to wait 1-2 weeks before each switch to shorter rods and to revert to longer rods if pain appears
- to adjust in increments of 0.5mm in the final phase of titration

To change rods, go to:  
[greatlakesdentaltech.com/panthera-d-sad](http://greatlakesdentaltech.com/panthera-d-sad)

### TITRATION TRACKING

Date	Rods Used
_____	_____
_____	_____
_____	_____
_____	_____