

smilesaver®

DIRECTIONS
FOR USE

Directions for Use

Smilesaver® is used for cleansing and freshening removable oral appliances. It is a clean product that provides a pleasant taste, is simple to use, portable, and totally convenient.

Removable Oral Appliances include Clear Aligners for teeth correction, retainers, dentures, night guards and other sleep disorder apparatus, and athletic mouth guards. Research has shown that all these appliances become harbors for significant amounts of bacteria that causes odors, bad taste, and a less-than-confident smile.

Directions:

1. Remove the oral appliance from the mouth.
2. Hold the device away from the body, with the pump sprayer 4 - 6 inches away from the device.
3. Completely depress the pump spray head at least twice and coat one side of the device with full sprays; then do the same on the other side of the device.
4. Wait 30 seconds, then re-insert the device into the mouth for its intended purpose. Or place in its storage case.
5. Spray all dental devices as often as needed to help eliminate up to 99.9% of bacteria*, stop odors and bad breath and get a fresh taste in your mouth.

Use the product only as directed. Keep product at ambient conditions in a clean, dry place, where it is always ready for use.

**As shown in laboratory testing*

