


# Telescopic Herbst® Advancement Instructions

- Allows the clinician to advance the mandible more gently and precisely without the use of shims.
- One turn of the protrusion nut on the lower plunger near the bicuspid (until the next hole appears) advances the mandible in 1/16mm increments.

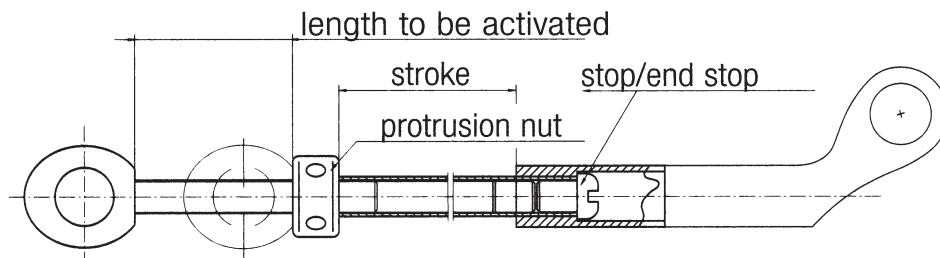

Advancements: 4 Turns = 1/4mm | 16 Turns = 1mm.

**Note: Arrows indicate the direction of advancement:**

Patient Right Side - Up | Patient Left Side - Down

- This unique telescopic hardware will not separate and allows an additional 5mm of movement. The lab standard is to reduce that advancement to 4mm, providing the clinician a 1mm retrusion allowance.
- Please contact our design specialist and set up a Master Rx for your appliances if you prefer otherwise.

